


## ⚠ Instructions for use - **The Exsanguinating Digit Tourniquet**

**Application** - Do not use if product packaging is damaged or open.

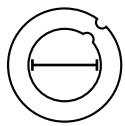


### TCS-1001

(yellow)

Smallest digits

Diameter: .85cm

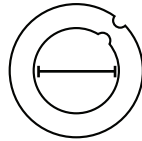


### TCM-2001

(red)

Small adult digits

Diameter: 1cm

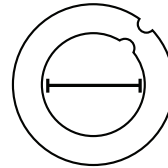


### TCL-3001

(green)

Large adult digits

Diameter: 1.25cm




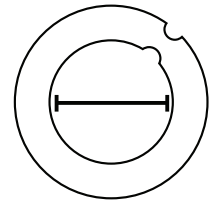


### TXL-4001

(blue)

Largest digits

Diameter: 1.5cm

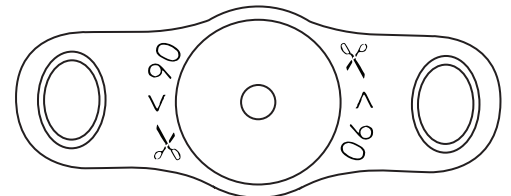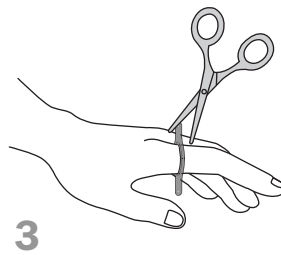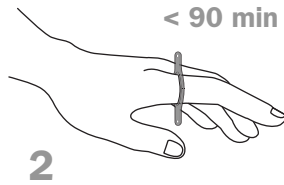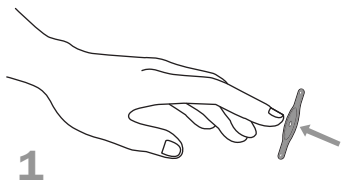

### TCU-6001 (Uni-Cot)

(blue)

All digits

## Specifications



No Latex



STERILE EO

15°C



35°C

# MAR-MED

Phone: 1.800.369.3434

Fax: 616.459.5119


Web: [www.marmedco.com](http://www.marmedco.com)

Email: [info@marmedco.com](mailto:info@marmedco.com)

**Mar-Med Co.**

333 Fuller N.E.

Grand Rapids, MI 49503

|  |  |  |   |
|--|--|--|---|
| <p><b>English</b><br/>See instructions for use.<br/>Important information.</p> <p> See instructions for use.<br/>Important information.</p> <p> Product does NOT contain latex.</p> <p>Excessive duration or pressure may cause tissue injury</p> <p> Re-use can cause infection</p>                               | <p><b>Spanish</b><br/>Ver instrucciones de uso.<br/>Información importante.</p> <p>El producto no contiene latex.</p> <p>La duración o la presión excesiva puede causar daño tisular</p> <p>La reutilización puede provocar infecciones</p>    | <p><b>German</b><br/>Siehe gebrauchsanweisung.<br/>Wichtige Informationen.</p> <p>Das product enthält keine Latex.</p> <p>Überlange Dauer oder Druck kann Gewebeverletzung</p> <p>Wiederholte Anwendung kann zu Infektionen führen</p> | <p><b>French</b><br/>Lire instructions avant usage. Renseignements importants.</p> <p>Ce produit ne pas contient du latex.</p> <p>Durée ou une pression excessive peut provoquer des lésions tissulaires</p> <p>La réutilisation peut provoquer une infection</p> |
| <p><b>Italian</b><br/>Veder il manuale delle istruzioni per l'uso.</p> <p> Veder il manuale delle istruzioni per l'uso.</p> <p> Informazioni importanti. Il prodotto contiene no lattice.</p> <p>Eccessiva durata o la pressione possono causare lesioni ai tessuti</p> <p> Se riusato può causare infezioni</p> | <p><b>Dutch</b><br/>Zie gebruiksaanwijzing.<br/>Belangrijke informatie.</p> <p>Dit product bevat geen latex.</p> <p>Buitensporig lange duur of druk kunnen veroorzaken weefselbeschadiging</p> <p>Hergebruik kan een infectie veroorzaken</p>  | <p><b>Swedish</b><br/>Se bruksanvisningen. Viktig information</p> <p>Produkten innehåller nej latex</p> <p>Orimligt lång tid eller tryck kan orsaka vävnadsskada</p> <p>Återanvändning kan orsaka infektion</p>                        | <p><b>Danish</b><br/>Se brugsanvisningen. Vigtig information.</p> <p>Produktet indeholder nej latex.</p> <p>Uforholdsmæssig lang varighed eller tryk kan fremkalde vævsskade</p> <p>Genbrug kan forårsage infektion.</p>  |
| <p><b>Finnish</b><br/>Katso käyttöohjeita. Tärkeää tietoa.</p> <p> Katso käyttöohjeita. Tärkeää tietoa.</p> <p> Tuote sisältää ei lateksia.</p> <p>Liiallinen kesto tai paine voi aiheuttaa kudonsvaurioita</p> <p> Uudelleenkäyttö voi aiheuttaa infektion</p>  | <p><b>Portuguese</b><br/>Veja as instruções de uso. Informações importantes.</p> <p>Este produto no contém latex.</p> <p>Quanto à duração excessiva ou a pressão pode causar lesão tecidual</p> <p>A reutilização poderá provocar infecção</p> | <p><b>Norwegian</b><br/>Se bruksanvisningen. Viktig informasjon</p> <p>Produktet inneholder ikke lateks</p> <p>Overtrykk eller varighet kan forårsake vevsskade</p> <p>Gjenbruk kan forårsake infeksjon</p>                            | <p><b>Greek</b><br/>Δείτε τις οδηγίες χρήσης. Σημαντικές πληροφορίες.</p> <p>Το προϊόν περιέχει όχι λάτεξ.</p> <p>Υπερβολική διάρκεια ή πίεση μπορεί να προκαλέσει τραυματισμό των ιστών</p>  |