

# Uni-Cot® makes it safe and easy to achieve hemostasis on digits.

Emergency physicians, hand surgeons, and podiatrists can trust the professionally styled Uni-Cot® for any type of case on any size digit.

### Single Size

Uni-Cot® is simple and easy to use compared to other products and risky improvised methods.

### Quick Hemostasis

Reliable pressure provides a visible field for effective treatment.

### Easy Application

Use one or two hands to apply the Uni-Cot® and stretch over wounds if necessary.

### Minimize Risks

Bright color and conspicuous shape makes it almost impossible to forget on a digit.

### Low Cost

Price comparable to sterile surgical gloves. \*Data on file



Method	Easy to Use	Immediate hemostasis	Professional appearance	Safe, even pressure	Brightly colored	Universal for fingers and toes
<b>Uni-Cot®</b>	✓	✓	✓	✓	✓	✓
Penrose Drain						
Rolled Surgical Glove						
Rubber Band						
Pneumatic Tourniquet			✓	✓		
Plastic Ring	✓	✓	✓	✓		

## The Uni-Cot® Makes a Difference

- Digit injuries commonly have high complication rates
- 38% of Foreign Bodies are missed on first visit (EM Clinics NA, 2007)
- Facilitates the use of tissue and adhesives, simplifying care and reducing visits

“It is critical to wound management” that “the wound is carefully explored in a bloodless field obtained through careful tourniquet application”

*(ACEP, Emergency and Primary Care of the Hand)*


“Use only CE marked tourniquets which are labeled and/or brightly coloured to enhance visibility. Surgical gloves should not be used for this purpose”

*(UK National Patient Safety Agency, 2009)*


## Indications

- Digit lacerations and avulsions
- Exploration of foreign bodies and deeper
- Nail and nail bed injuries
- Elective procedures


## Apply with 1 or 2 Hands

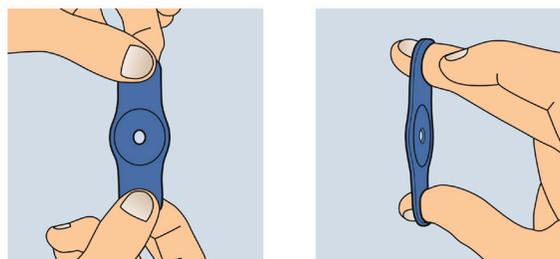


## Stretches over Wounds

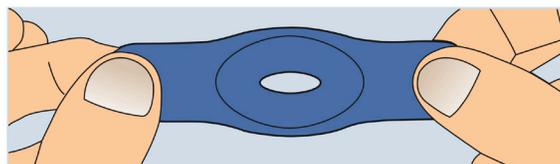


## Using the Uni-Cot® is Easy for Fingers and Toes

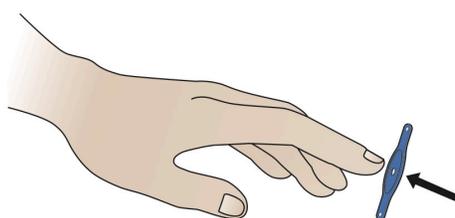

1) Slide proximally to apply and exsanguinate.

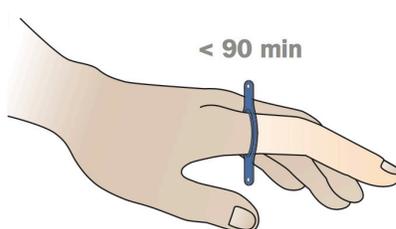

2) Even compression is achieved once in place

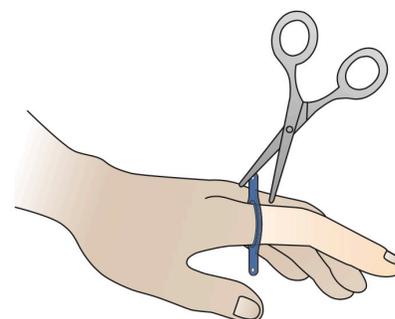

3) Remove by severing the exterior.

## Place Your Order Today

To learn more about the Uni-Cot® or to place an order, contact Mar-Med at **1.800.369.3434** or **info@marmedco.com**

**MAR-MED**

333 Fuller Ave., NE, Grand Rapids, MI 49503 USA

800.369.3434  
616.459.5119 Fax

info@marmedco.com  
www.marmedco.com